



Tip for use of FAQ: Click on questions to go to answer.

Please also go to [www.resound.com/control](http://www.resound.com/control) for more user guidance.

GETTING STARTED.....	2
In which countries is the app available on the App Store or Google Play? .....	2
Why do I not see the app on Google Play on my Android phone? .....	2
Which hearing devices do I need to use the app? .....	2
Why do I need the ReSound Unite Phone Clip+ to use the app?.....	2
I have an original ReSound Unite Phone Clip. Will it work with the app? .....	2
How can I tell if I have a Phone Clip or a Phone Clip+?.....	3
Which mobile device do I need to use the app? .....	3
Which operating system must my mobile device have?.....	3
How do I pair my mobile device to my Phone Clip+?.....	3
I've downloaded the app and paired my mobile device to my Phone Clip+. Now what do I do?.....	3
I've downloaded the app but I can't find it on my phone? .....	4
Did you remember to disable your phone's lock sounds, keyboard clicks and other touch sounds? ..	4
Why does my app look different from what I see on the web landing page?.....	4
VOLUME & MUTE .....	4
What is the difference between the white and the blue volume bars? .....	4
Can I mute the sound?.....	4
Can I mute the sound on ONLY my right or left hearing aid? .....	5
Why do I not see the mute function on my app? .....	5
How do I make volume adjustments to ONLY my right or left hearing aid? .....	5
How do I reset volume to default settings?.....	5
Can I use the app to adjust phone volume during calls or while streaming music? .....	5
Can I use the app to adjust hearing aid volume during calls or while streaming music? .....	5
What are the small 'punch holes' on the volume bars?.....	5
PROGRAM.....	6
Why can I not change programs while I'm listening to music or am on a phone call? .....	6
Why can I not change programs at all? .....	6
Why do my hearing aids change programs at message alerts, keyboard use and locking the phone? ..	6
If I edit the names of the programs, will my Hearing Care Professional be able to recognize them? ....	6
Why did I lose my personalized program names that I entered? .....	6
Why did the name of the program change when I chose it? .....	6
CONNECTION .....	7
I have turned on the Phone Clip+ but my mobile device is not making the Bluetooth connection.....	7
My phone is connected via Bluetooth to the Phone Clip+ but the app says it is not connected to the hearing system?.....	7
What do the lines on the Link screen mean? .....	7
What is the red exclamation mark on the Link button trying to tell me?.....	7
OTHER.....	8
Can I get the app in a language other than English? .....	8
What is Demo mode? .....	8
How do I remove the app?.....	8



### GETTING STARTED

#### **In which countries is the app available on the App Store or Google Play?**

A: ReSound Control is available in all world markets except: Bulgaria, Croatia, Estonia, Latvia, Lithuania, Slovakia and Slovenia.

[Back to top](#)

---

#### **Why do I not see the app on Google Play on my Android phone?**

A: There can be two reasons:

1. Some Android phones and/or operating systems will not be compatible with the ReSound Control app. If you do not see the app on Google Play it is because the app is not compatible with your phone model and/or your operating system. Minimum requirement for operating system is 2.3.5.
2. The app is not available in your country. Please see question '[In which countries is the app available on the App Store or Google Play?](#)'

[Back to top](#)

---

#### **Which hearing devices do I need to use the app?**

A: You need ReSound wireless-enabled hearing aids and the ReSound Unite™ Phone Clip+ to use the app. Please note that in order to use the app with wireless ReSound Alera™ hearing aids, the remote control functionality on the Phone Clip+ must be enabled by initially pairing the hearing aids with a ReSound Unite Remote Control. Please consult your Hearing Care Professional for assistance.

[Back to top](#)

---

#### **Why do I need the ReSound Unite Phone Clip+ to use the app?**

A: The mobile device communicates via Bluetooth to the Phone Clip+ just like other Bluetooth headset devices. The Phone Clip+ then uses the GN ReSound proprietary 2.4GHz wireless technology to communicate with the hearing aids.

[Back to top](#)

---

#### **I have an original ReSound Unite Phone Clip. Will it work with the app?**

A: While the Phone Clip remains a valuable phone accessory, only the Phone Clip+ works with the app.

[Back to top](#)

---



### How can I tell if I have a Phone Clip or a Phone Clip+?

A: The Phone Clip+ has additional functionality. That means that you will find mute and program change buttons on the front of the Phone Clip+ device. Also, the clip on the Phone Clip+ is a swivel clip whereas the one on the original Phone Clip is fixed. The Phone Clip+ was released Dec 2012.

[Back to top](#)

---

### Which mobile device do I need to use the app?

A: iPhone/iPad/iPod touch or Android smartphone/tablet. Please note that the app is not compatible with all Android devices.

[Back to top](#)

---

### Which operating system must my mobile device have?

A: iOS: Operating system 7.0 or later. Android: Operating system 2.3.5 or later. Please note that the app is not compatible with all Android devices even if the requirements for operating system are fulfilled.

[Back to top](#)

---

### How do I pair my mobile device to my Phone Clip+?

A: Inside the app, you will find pairing guidance under Settings. You can also go back to [www.resound.com/control](http://www.resound.com/control) and download the Quick Guide for pairing guidance.

[Back to top](#)

---

### I've downloaded the app and paired my mobile device to my Phone Clip+. Now what do I do?

A: Simply tap the launch icon on the home screen of your mobile device to activate the app. An entry flow will begin:

- Step 1: Do you want to connect now or run in demo mode? Choose connect now.
- Step 1a: (Only on Android) Required devices: We inform you that you need ReSound wireless hearing aids and the ReSound Unite Phone Clip+ (released Dec. 2012) to operate the app.
- Step 2: Terms of use: We inform you that you should familiarize yourself with the printed user guide for the Phone Clip+. It is necessary to accept the terms of use to proceed.
- Step 3: We recommend that you disable any lock sounds, keyboard clicks and other touch sounds on your mobile device to improve your user experience. This is usually done under Settings and Sound.
- Step 4: You're ready to run ReSound Control. If you need any guidance on how the app works or how to pair to the Phone Clip+, that information is available under Settings.

[Back to top](#)

---



**I've downloaded the app but I can't find it on my phone?**

A: You will find ReSound Control listed with your other apps under 'Control'.

[Back to top](#)

---

**Did you remember to disable your phone's lock sounds, keyboard clicks and other touch sounds?**

A: Whenever the Phone Clip+ detects that there is a sound from the mobile device, it assumes you will want to hear that sound in your hearing aids so it automatically changes to a phone streaming program. It will change right back when the sound stops. You can limit this by disabling unnecessary sounds, for example lock sounds, keyboard clicks and other touch sounds (disable under Settings on your mobile device) and you can also set text message and other alerts to Vibrate Only, not tone.

[Back to top](#)

---

**Why does my app look different from what I see on the web landing page?**

A: All images on the landing page show the iPhone version of ReSound Control. On Android, the app might look a little different in order to stay true to that platform. The appearance is also dependent on how each Android phone is designed. All basic functions are the same.

[Back to top](#)

---

## VOLUME & MUTE

**What is the difference between the white and the blue volume bars?**

A: The white volume bars are for your hearing aid microphones. The blue volume bars are for your streaming device, ReSound Unite TV Streamer 2 or Mini Microphone, and/or for your phone calls and phone music. On ReSound Alera devices you will only see one kind of volume bars – they will adjust both hearing aid and streamer volume at the same time.

[Back to top](#)

---

**Can I mute the sound?**

A: Yes, you can mute the sound from the hearing aid microphones. Just tap the mute button. Your options for muting the sound from a streaming device are to either turn the volume down or to choose a non-streaming program.

[Back to top](#)

---



### Can I mute the sound on ONLY my right or left hearing aid?

A: Yes, just slide right or left on the volume screen and press the 'half' mute button.

[Back to top](#)

---

### Why do I not see the mute function on my app?

A: If you have ReSound Alera wireless hearing aids, the mute function is not available.

[Back to top](#)

---

### How do I make volume adjustments to ONLY my right or left hearing aid?

A: Slide right or left on the volume screen and make your desired adjustment. Please note, if you have wireless hearing aids where volume synchronization is activated, your hearing aids will always synchronize volume. If you would like to be able to make single-sided volume adjustments on the app, please talk to your Hearing Care Professional at your next visit.

[Back to top](#)

---

### How do I reset volume to default settings?

A: Either slide the volume bars up or down until the 'punch holes' are visible in the top white bar or re-choose the program on the program selection screen.

[Back to top](#)

---

### Can I use the app to adjust phone volume during calls or while streaming music?

A: Yes, with most ReSound wireless hearing aids, you can use the blue volume bars on the volume screen to adjust the volume level. With ReSound Alera hearing aids, you will be adjusting the volume of both the hearing aids and the streamed sound.

[Back to top](#)

---

### Can I use the app to adjust hearing aid volume during calls or while streaming music?

A: Yes, with most ReSound wireless hearing aids, you can use the white volume bars to adjust the hearing aid volume level. With ReSound Alera hearing aids, this functionality is not available.

[Back to top](#)

---

### What are the small 'punch holes' on the volume bars?

A: The 'punch holes' indicate your default volume setting.

[Back to top](#)

---



### PROGRAM

#### **Why can I not change programs while I'm listening to music or am on a phone call?**

A: When you receive sound from your mobile device, the hearing aids automatically move to a streaming program. In order to go back to a hearing program, please exit the active phone operation.

[Back to top](#)

---

#### **Why can I not change programs at all?**

##### **Why do my hearing aids change programs at message alerts, keyboard use and locking the phone?**

A: Whenever the Phone Clip+ detects that there is a sound from the mobile device, it assumes you will want to hear that sound in your hearing aids so it automatically changes to a phone streaming program. It will change right back when the sound stops. You can limit this by disabling unnecessary sounds, for example lock sounds and keyboard clicks (disable under Settings on your mobile device) and you can also set text message and other alerts to Vibrate Only, not tone.

[Back to top](#)

---

#### **If I edit the names of the programs, will my Hearing Care Professional be able to recognize them?**

A: Yes. Editing names of programs is only stored in the app. When your Hearing Care Professional reconnects your hearing aids to the fitting software in his/her office, the fitting software will overwrite the changes you made. If you want to keep the names you edited, either ask your Hearing Care Professional to enter them in the fitting software or edit them again after your visit.

[Back to top](#)

---

#### **Why did I lose my personalized program names that I entered?**

A: Please see answer above.

[Go to question above](#)   [Back to top](#)

---

#### **Why did the name of the program change when I chose it?**

A: If your Hearing Care Professional has entered names of your programs in your hearing aids that are not default from the manufacturer, you will see these names after the first time you enter a program.

[Back to top](#)

---



### CONNECTION

**I have turned on the Phone Clip+ but my mobile device is not making the Bluetooth connection.**

A: Did you pair the Phone Clip+ with your mobile device? If not, please see info on the web landing page for how to get started ([www.resound.com/control](http://www.resound.com/control)). If you did pair the two devices, connection should happen automatically. If not, go to the Bluetooth settings of your mobile device and click 'Hearing Aid Phone' to connect. Most mobile devices will display a small Bluetooth or headset icon in the top status bar when actively connected.

[Back to top](#)

---

**My phone is connected via Bluetooth to the Phone Clip+ but the app says it is not connected to the hearing system?**

A: In rare cases it may happen that the app cannot detect the Bluetooth connection established between the phone and the Phone Clip+. Please try to turn off the Phone Clip+, wait three (3) seconds, and then turn it on again.

[Back to top](#)

---

**What do the lines on the Link screen mean?**

A: Green line: connection established. Red line with exclamation mark: connection missing. Press mark/line for troubleshooting. Grey dotted line: Pairing of streaming device exists but the device is not currently connected. Turn on device to connect to hearing aids.

[Back to top](#)

---

**What is the red exclamation mark on the Link button trying to tell me?**

A: You are likely to be missing a connection between your mobile device and your hearing system. Tap the button and you will see an overview of the connections between your mobile device, your Phone Clip+, your hearing aids and your streaming device (if you have a streaming device paired in). If a particular connection is not established, the line will be red and have an exclamation mark on it. Tap the exclamation mark or the red line to get some tips for reestablishing the connection.

[Back to top](#)

---





### OTHER

#### **Can I get the app in a language other than English?**

A: Yes, the app is also available in Czech, Danish, Dutch, Finnish, French, German, Greek, Hungarian, Italian, Japanese, Korean, Norwegian, Polish, Portuguese, Romanian, Spanish, Swedish, Chinese (simplified + traditional).

[Back to top](#)

---

#### **What is Demo mode?**

A: Enabling Demo mode means that you can run the app without connection to the hearing system for demonstration purposes or just to try it out. You can enable/disable Demo mode under Settings.

[Back to top](#)

---

#### **How do I remove the app?**

A: On an Apple device: Put your finger on the launch icon on the home screen and keep it there until it starts 'shaking' and a little 'x' appears. Tap the 'x' to delete the app. Confirm deletion. Then press the Home button to return your home screen to normal.

On an Android device: Ways to remove apps vary on Android devices – please consult your smartphone user guide.

[Back to top](#)

---